

# Synapse Mobility client quick start for IOS devices

The Synapse Mobility thin client server is provided to all Synapse PACS users by the PACS Workgroup, IUH Radiology Department

From App Store download and install: "Fujifilm Synapse Mobility"

Open the Mobility App and enter the following information:

**Server name:**  
iuhynmobility

**Address:**  
iuhynmobility.iuhealth.org:8443

**Secure Conn:**  
ON

## SUPPORT:

For acute support contact the HelpDesk at phone: 962-2828.

For comments and questions contact the RIS Team at risteam@iuhealth.org




**Log in:**  
Enter your MHG Username and Password



**1**  
Enter demographics (one or more)



**2**  
Select "Modality"  
Select "Date within last"

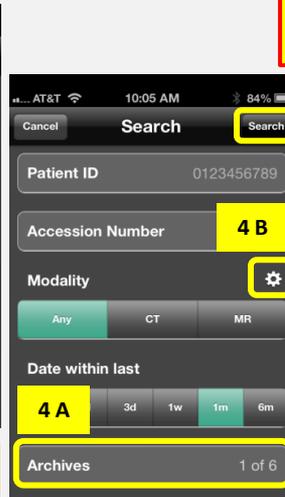


**3**  
Scroll on screen down to bottom: "Archives"

**5.**  
Click "Search"



**4**  
A. Select Archive(-s) (Mandatory)  
B. Specify more Modalities (Optional)

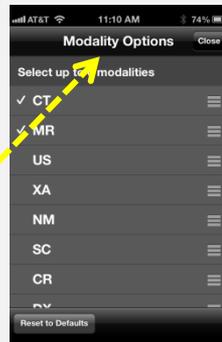


**6**  
A. Select sort order  
B. Select exam to view




**7**  
A. Select report and/or  
B. Select series




**8**  
**Review**  
A. Scroll (one finger)  
B. Zoom (pince)  
C. Pan (two fingers)  
D. WW/WL (Tap + finger)

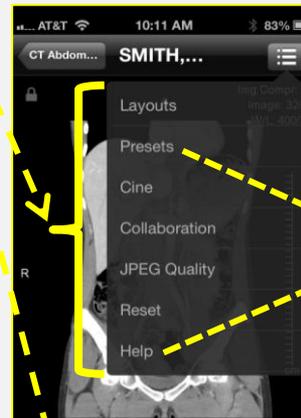

Select added functions shown above and examples on the right)

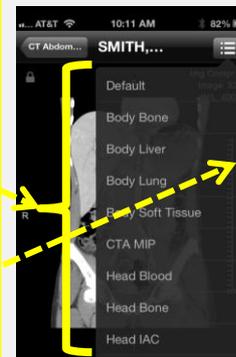

Select WW/WL

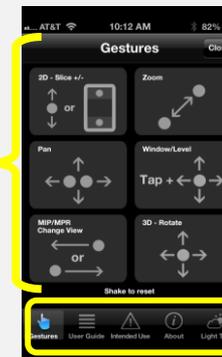

Please, note:  
More functions available here!

Additional access via:  
Desktop and laptop  
Mac or PC

Internet browser access available at:  
<https://iuhynmobility.iuhealth.org:8443/pureweb/synapse/studybrowser>  
Use MHG Username and Password for log in. VPN is not necessary!